## WHEN SHOULD I CALL MY DOCTOR?

- If you experience any of the following symptoms please contact your physician immediately at the number listed on your Patient Information Card:
- Fever, rash, bleeding, wound drainage, persistent tenderness or swelling in the groin, redness and/or warmth to the touch, numbness, tingling or pain in the extremity when walking, bruising at the puncture site, other unusual symptoms.

For the complete Instructions for Use and explanation of symbols provided on your Patient Information Card, refer to :  
<http://www.terumomedical-labeling.com>.

The SSCP for the FemoSeal VCS medical device can be found at:  
<https://ec.europa.eu/tools/eudamed>.

**Terumo Corporation**  
+81333748111

**Terumo Europe NV**  
+3216381211

**Terumo  
Interventional Systems EMEA**  
+331471609 30

*EMEA Sales Offices*

**TerumoEurope NV  
Africa Business Division**  
+32 16 38 13 08

**Terumo Europe NV  
Benelux Sales Division**  
Belgium:  
0800 14468  
The Netherlands:  
0800 0231938

**Terumo Europe NV  
Emerging Market Division**  
+3216381211

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® Registered Trademark  
Published by Terumo Europe NV

15699GB06211T111

[www.terumo-europe.com](http://www.terumo-europe.com)

A PATIENT'S GUIDE TO

**FemoSeal™**  
Vascular Closure System

# ACTIVE VASCULAR CLOSURE

Your physician has decided to use FemoSeal™ vascular closure system. This guide will provide you with further information about FemoSeal™ and will give you some guidelines for your return home.

• Implantable components are MRI safe.

## HOW DOES FEMOSEAL™ WORK?

FemoSeal™ is made of three bioabsorbable components: an inner seal, an outer locking disc and a suture. A delivery system allows the physician to guide the inner seal into the artery through the hole created during the procedure. The inner seal is drawn against the wall of the artery while the suture allows the outer locking disc to lock against the outer wall of the artery to create a secure seal over the entry point<sup>1</sup>. A sterile dressing is then applied to the site. All three components dissolve and are fully absorbed into your body<sup>2</sup>.

*The inner seal and outer locking disc achieve hemostasis and are held together by a bioabsorbable suture.*

## HOW SHOULD I CARE FOR THE SITE WHEN I RETURN HOME?

The following are guidelines suggested by Terumo for post-procedure site care and activities<sup>3</sup>.

**Note:** These materials are not intended to replace your doctor's advice. For any questions or concerns you have regarding the medical procedures, devices and/or your personal health, please discuss with your physician. After you are discharged from the hospital, you should modify your activities.

- You may shower, but do not take a bath, use a hot tub or swim until the skin site is healed<sup>3</sup>.
- For 48-72 hours, you should refrain from straining or lifting anything over 4.5 kg<sup>3</sup>.
- Avoid driving on the day of your discharge<sup>3</sup>.

Care for the wound as directed.

- It is normal to feel a small lump, about the size of a pea, and/or note mild tenderness in the groin area<sup>3</sup>.
- Discomfort is common during the healing process, after intravascular procedures<sup>3</sup>.
- After 24 hours, remove the dressing. Gently clean the site with mild soap and water. Dry the area and cover it with an adhesive bandage. Change the bandage if it becomes soiled or wet. Cover the area daily with a new bandage until the skin heals<sup>3</sup>.
- An implant card was provided by your physician post procedure. Please keep this card with you for the next 18 months.

1. FemoSeal Instructions for Use.

2. Holm et al. Randomised comparison of manual compression and FemoSeal™ vascular closure device for closure after femoral artery access coronary angiography: the closure devices Used in everyday Practice (CLOSE-UP) study. Eurointervention 2014;9:183-190.

3. Hamel et al. Femoral artery closure after cardiac catheterization. Critical Care Nurse Vol 29, No. 1, Feb. 2009.

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© Rekisteröity tavaramerkki
**Terumo Sweden AB**
**Terumo UK Ltd**